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ENG

Thank you for buying our product! To ensure safe and successful use, carefully read this user manual before operating the wheelchair. Incorrect use may result in health damage, injury, or accidents.

NOTE: Check all parts for damage caused during transit. If such damage is noticed, DO NOT use the product.

GENERAL WARNINGS

Read and understand the instructions before assembling or using the product. If unsure, consult a healthcare professional, retailer, or technician to avoid injury or damage. Consult your doctor or therapist for appropriate adjustments and use. Before use, ensure all adjustments are secure and check the condition of wheels and moving parts. Keep all castors in contact with the ground during use to prevent wheelchair tipping and user injury.

WARNING: During use, assembly, and adjustments, be cautious to avoid trapping body parts in openings between components. Stabilize the position after adjustments by securely tightening nuts/bolts.

NOTES:

In case of a product-related 'serious incident' that has, could have, or may lead to the following:

- (a) the death of a patient, user or other person; or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health

Report such a 'serious incident' to both the manufacturer and the relevant authority in the user's or patient's residing Member State. In Poland, contact the Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products.

NOTES: If you experience pain, allergic reactions, or unclear symptoms related to the medical device, consult a healthcare professional.

ATTENTION: The manufacturer is not liable for damage due to negligent maintenance, inadequate servicing, or failure to follow manual instructions.

NOTE: There may be a risk of tipping over if the product is used incorrectly. Please follow the instructions for getting on/off/moving.

ATTENTION: it is forbidden to use the product other than for its intended purpose

PLEASE NOTE: this product is not permitted for use by children under 3 years of age.

COMMENTS AND WARNINGS:

- When getting in or out of a wheelchair, make sure the brakes are on.
- Care must be taken when negotiating kerbs and making uphill turns.
- Steep and uneven surfaces should be avoided as this can cause the wheelchair to tip over.
- Move the wheelchair smoothly in all conditions.
- Keep both feet on the footrests at all times, and ensure that clothing is properly tucked and does not block the wheels.
- Always drive carefully on hills.
- Always keep both hands on the push handles for proper control of the wheelchair (applies to the helper).
- Keep the wheelchair away from sources of fire
- Under no circumstances should the wheelchair be overloaded.
- Do not hang any heavy objects on the back of the wheelchair as this may affect the stability of the product especially when climbing hills.
- The wheelchair can only be used by one person.
- Do not lift the front wheels while driving as this can cause a fall and serious injury

PURPOSE: The product is designed to comfortably transport people with limited or no walking ability. The wheelchair is designed to transport one person.

INDICATIONS: Contradictions to sitting position. Conditions needing upper spine and head stabilization. Do not leave patients with mental disorders unattended. Physical or mental limitations preventing safe device use.

FEATURES:

- Steel frame
- Easy-to-clean upholstery
- Removable table
- Wheelchair foldable

USE OF THE PRODUCT:

Use the wheelchair on flat surfaces or indoors; avoid outdoor use in bad weather, on mud, snow, sand, gravel, and uneven surfaces. Steer clear of heavy traffic and sunlight exposure to prevent wear and tire damage. Follow road safety regulations when using on roads; stick to pavements and pedestrian crossings. When not in use, store the wheelchair in a cool, dry place, avoiding direct sunlight, wind, rain, fog, and water. Avoid placing heavy objects on the wheelchair.

NOTE: THE PHOTOS USED TO PRESENT THE FUNCTION OF THE WHEELCHAIR ARE FOR ILLUSTRATIVE PURPOSES ONLY, THEY MAY DIFFER FROM THE ACTUAL APPEARANCE OF THE PRODUCT.

DISPLAY

Unfold the wheelchair by extending the seat outward, pressing it down at the sides until fully unfolded. Then, unfold the backrest, attaching it to the corresponding holes at the top. Finally, secure the head cushion with Velcro (refer to Fig. 1 and 2).

FIG. 1

FIG. 2

FIG. 3

FIG. 4

FIG. 5

5

SUBMISSION

For transport, remove the high backrest and fold down the lower backrest. Lift the seat's center upward to fold the wheelchair (refer to Figures 6 and 7). For complete folding, first, remove the footrests, then follow the steps outlined above.

FIG. 6

FIG. 7

FOOTSTOOLS

Fit the footrests by placing them in the appropriate holes and turning them inwards to lock into position (refer to Fig. 8). After fitting, check stability by pulling. To remove, pull the handle backward (Fig. 9), turn the footrest outward, and pull it out. To completely remove, lift it up (Fig. 10).

FIG. 8

FIG. 9

FIG. 10

FOOTREST ANGLE ADJUSTMENT

To adjust the angle, raise the footrest by gripping the metal tube from underneath (Fig. 11). For a downward adjustment, press the adjustment lever (Fig. 12). Once the correct leg position is set, release the adjustment lever (Fig. 13).

FIG. 11

FIG. 12

FIG. 13

HEIGHT ADJUSTMENT OF THE FOOTRESTS

Loosen the footrest plate screw using a hex key, adjust the footrest height, then tighten the screw (Fig. 14 and 15). Calf support height can also be adjusted in three steps (Fig. 16).

FIG. 14

FIG. 15

FIG. 16

BACKREST ANGLE ADJUSTMENT

The caregiver can adjust the backrest angle using the red levers at the handles (refer to Fig. 17 and Fig. 18).

FIG. 17

FIG. 18

Table assembly and disassembly

Once unfolded, attach the table holders to the frame tubes on both sides. Do not tighten the screws; insert the two tubes into the table's corresponding holes (refer to Fig. 19). Slide the table into the latch hole and tighten the screws to the maximum. When lifting, pull the table upward (Fig. 20).

FIG. 19

FIG. 20

Brake user manual

Apply the brakes each time you enter or exit the wheelchair. Hold the brake handle and move it forward (unlocking) or backward (locking). Once locked, you can safely enter or exit.

NOTE: Avoid using the brake when descending a hill to prevent overturning. When using the handbrake, grip the lever with both hands, adjusting the grip to control braking force or wheelchair speed.

NOTE: The wheelchair is also equipped with brakes for the carer.

Quick-folding backrest function

The fold-back function allows the wheelchair to be stored in narrow spaces. Press the handles with both hands and tilt the push handles to fold. To open, push the handles upward until they lock.

PROBLEM SOLVING

Problem	Control and problem solving
1. wheelchair changes direction of travel to the right or to the left	<ul style="list-style-type: none">• The condition of the wheels should be checked.• Check the angle of the caster bracket• Please check that the rear wheels are evenly spaced from the seat frame.
2. Reduced turning or driving speed	<ul style="list-style-type: none">• Check the tightness of the bolts and nuts.• Check that the wheel turns normally
3. Wheel problems - Wobbling	<ul style="list-style-type: none">• Check the tightness of the bolts and nuts.• Check the angle of the caster bracket.
4. Creaking and other disturbing sounds	- Check the tightness of the bolts and nuts. Inspect the wheel for rust or other defects.
5. wheelchair gaps	- Check the tightness of the bolts and nuts.

TECHNICAL SPECIFICATIONS

Overall length (recumbent position)	1640mm	Seat depth	430 mm
Overall length (sitting position)	1189mm	Seat width	460mm
Length after folding	1189mm	Distance between armrests	480 mm
Overall width	700mm	Seat height	450mm
Width when folded	300mm	Front wheels	8'
Overall height	12000mm	Rear wheels	23'
Backrest height	450mm	Weight	22,2 kg
Backrest height (including head rest)	770mm	Maximum user weight	120 kg

CLEANING

Clean the frame, seat, backrest and other components with a damp cloth and mild detergent. When cleaning the wheelchair, always wipe it dry.

WHEEL MAINTENANCE

1. Clean the wheels with warm water and a mild detergent. Dry with a dry cloth.
2. Lubricate periodically with grease if the wheels start to squeak.

NOTE: STRONG DETERGENTS AND SHARP OBJECTS MUST NOT BE USED FOR CLEANING.

This sign indicates the maximum weight of the user.

STORAGE:

Store the product in a dry place, do not expose to direct sunlight

COMPOSITION

The frame is made of steel and the seat and back are made of polyurethane leather, with sponge inside.

SCOPE OF SUPPLY:

TROLLEY, TWO FOOTRESTS, TABLE, HEAD REST, INSTRUCTION MANUAL.

UTILISATION

Dispose of municipal waste in accordance with the applicable disposal conditions.

TARGET PATIENT GROUP

Individuals with conditions for which this device is intended (refer to instructions) can purchase it independently or based on a doctor's, therapist's, or specialist's recommendation. Regardless of the purchase method, consider available sizes, necessary functions, variants, indications, contraindications, and manufacturer-provided information.

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Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. AN TAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period AN TAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. AN TAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to AN TAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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